




Alco-Screen 02™ Alcohol Testing Kit

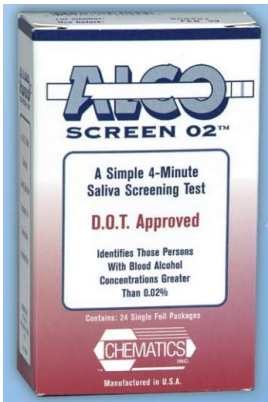

SPECIFICATIONS

Test Medium: Saliva

Test Principle: Enzyme

Reading Time: Four minutes

Results: Qualitative pass/fail

Measurement Range: 0.02%

Shelf Life: One year from date of manufacture



STEP 1

Place end of strip in mouth to collect saliva.



STEP 2

A green line appears within minutes to indicate the presence of alcohol at the .02% level or higher.

DOT Approved as Alcohol Screening Device

The Alco-Screen 02 was one of the first devices to be approved by the U.S. Department of Transportation (DOT) as an Alcohol Screening Device. DOT approval means that the Alco-Screen reliably and accurately detects alcohol at the 0.02 concentration or higher, and does not give false positive readings when no alcohol is present. DOT approval allows users to use the Alco-Screen 02 for screening tests required by Dept. of Transportation regulations.

Convenient, Fast Results

Testing with the Alco-Screen 02 is as easy as placing one end of the test strip in the subject's mouth for a few seconds to wet it with saliva. The operator reads results in four minutes.

Pass/Fail Results

Because the proportion of alcohol in saliva is directly related to the proportion of alcohol in blood, the Alco-Screen 02 can easily and quickly detect the presence of alcohol. Alcohol concentrations of 0.02 or greater cause a solid green line to form on the reagent pad. If no alcohol is present, the reagent pad remains unchanged.

STT Training Kit

DOT regulations require operators to be certified as a Breath Alcohol Technician (BAT) or a Screening Test Technician (STT) to perform DOT regulated alcohol testing with the Alco-Screen 02. The STT Training Kit provides a cost-effective and convenient way to accomplish this required certification training.

remedyNOW Consulting, LLC

4432 N 109th Street • Wauwatosa • WI • 53225

414-858-8014 • www.remedynowconsulting.com